Honeypatch Dry – tulle imprégné de miel médical (10x10 cm Art. No.: 1052153) (5x5 cm Art. No.: 1057269)

Miel médical

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Honeypatch Dry- tulle with medical honey (10x10 cm Art. No.: 1052153)

Medical honey

Chestnut honey is a rich source of vitamins and minerals, especially calcium, manganese, and potassium. Honey has a high amount of sugar (min. 80 %), low level of water (max. 17 %) and low pH value. All this characteristics lead to a high osmotic activity in the wound and these create discouraging environment for micro-organisms growth and at the same time stimulates wound healing.

Effect

Honeypatch Dry tulle is designed for wound bed protection. It is soft and thin, in combination with medical honey, it provides improved contact with wound bed. Porous structure of the polyester tulle allows the exudate to flow from the wound through to a secondary dressing; in addition, it prevents the dressing from sticking to the wound bed. The tulle has bacteriostatic effect; due to honey's high osmotic activity it creates discouraging environment for microorganism growth (osmotic pressure drives fluid out of the bacterial cells and so inhibits their growth) and at the same time stimulates wound healing. Osmosis process in the wound also stimulates the flow of the substances, which promote healing in the body itself. Naturally produced chestnut honey is a rich source of vitamins, enzymes, and minerals (especially calcium, manganese, and potassium), which additionally stimulate wound healing and shorten the healing period.

Advantages of using Honeypatch Dry

- Stimulated granulation and epithelialization
- Lower risk of wound infection
- No sticking onto the wound
- No pain at dressing change

Intended use

Honeypatch Dry tulle is designed for treating wound bed of a wide spectrum of wounds – from chronic to acute. It is particularly convenient for treatment of skin abrasions, wounds with weak secretion, and minor lesions. The tulle is not absorbent and therefore needs a secondary absorbent dressing or a plaster. By using the tulle, there is a possibility of only changing the secondary dressing, and thus protecting the wound bed and preserving the healthy, newly formed tissue.

Instruction for use

After opening the sterile pouch, remove protective transparent foil from both sides of the tulle. Place the tulle into the clean wound. Then place a secondary dressing, which will absorb the wound secretion over the mesh. Now fasten everything securely with a fastening bandage or adhesive tape. Mesh can be cut or fold.

At the beginning of healing process it is recommended to change the tulle every two days, for the effects of honey are decreased by dissolving in wound secretion. Later on, when the amount of the secretion decreases, Honeypatch Dry tulle can be changed every three to four days. In case of tulle drying and sticking to the wound, moisten it with saline solution.

Storage

Product must be stored between 5 and 35 °C, horizontally.

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