FR

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EN

HONEYPATCH Tube –STERILE MEDICAL HONEY • 20g

Medical honey

Chestnut honey is a rich source of vitamins and minerals, especially calcium, manganese, and potassium. Honey has a high amount of sugar (min. 80%), low level of water (max. 17%) and low pH value. All these characteristics lead to a high osmotic activity in the wound and these create a discouraging environment for micro-organisms growth and at the same time stimulates wound healing.

Which wounds are suitable for treatment with medical honey?

- deep wounds
- chronic and acute wounds
- surface wounds
- infected wounds
- burns
- diabetic ulcers

Instruction for use

After opening the tube, squeeze a small amount of honey on sterile swab and throw away. Apply medical honey onto sterile primary dressing like alginate dressing, nonwoven swab or PU foam and place on clean wound. In the case of a substantial excrete, put additional highly absorbent dressing and fasten everything with a fastening bandage. In the beginning of healing perform dressing every day, later could be done on two or three days, depends on amount of excrete.

Use the product within four months after first opening. Close the tube well after each usage. Apply honey from tube only onto sterile dressing, to prevent possible contamination of the remaining content in the tube. Product is sterile only after first opening. Do not use the product, if the tube is damaged.

Warnings

Although allergies to honey are rare, we warn those that have an established allergy

to honey or bee venom against the use of medical devices which contain honey.

Diabetes

Clinical tests of the healing of wounds with HONEYPATCH Tube medical honey with people with diabetes show very good experience and results. The treatment should nevertheless be medically controlled.

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